KENDALL**ARGYLE™****Jejunostomy kit****Jejunostomy kit**

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STERILE EO



™ Trademark of Tyco Healthcare Group LP or its affiliate.


TYCO HEALTHCARE GROUP LP, 15 HAMPSHIRE STREET, MANSFIELD, MA 02048 • MADE IN IRELAND

© 2010 TYCO HEALTHCARE GROUP LP • ALL RIGHTS RESERVED • www.tycohealthcare.com


TYCO HEALTHCARE U.K. LTD, GOSPORT, PO13 0AS, U.K.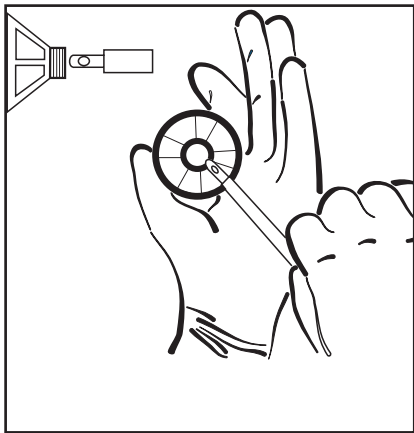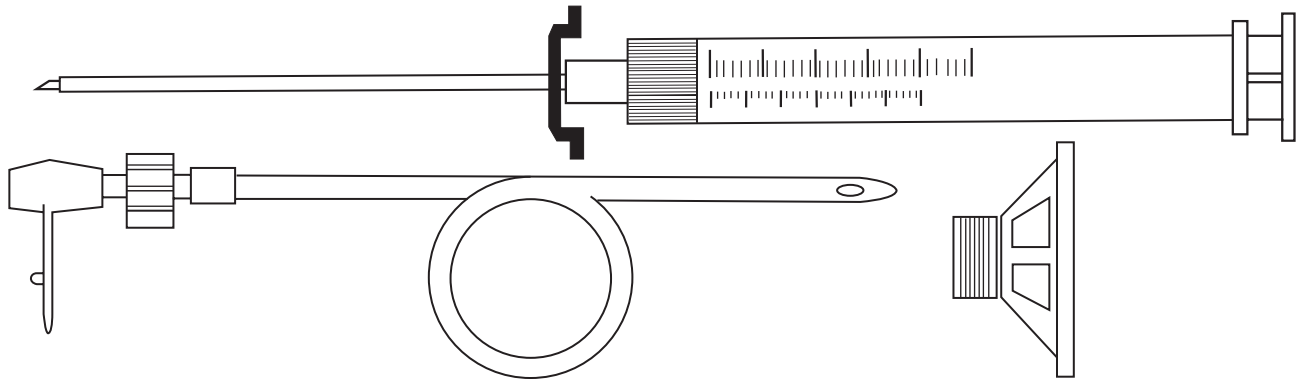


1

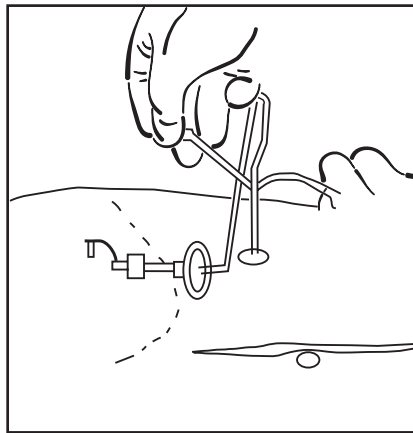
2

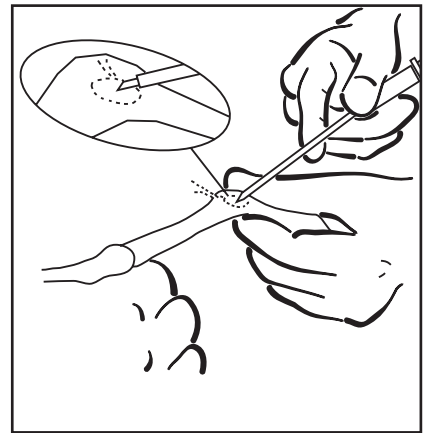
3

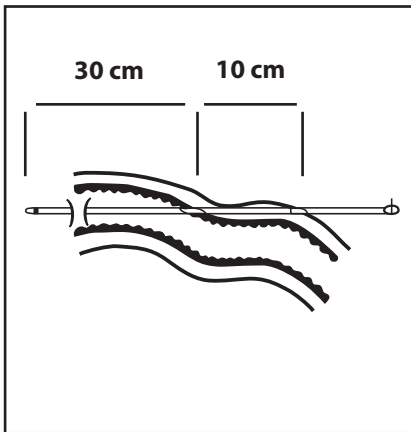4

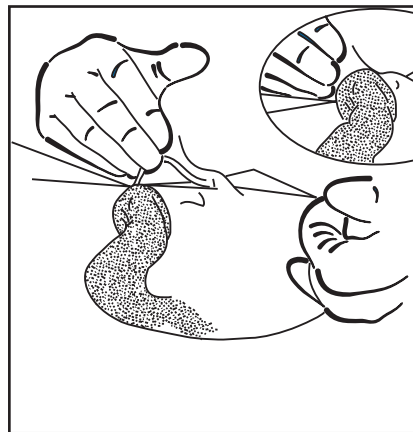5

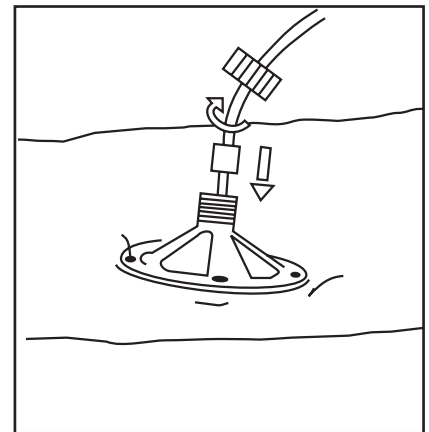6

INTRODUCTION

The Jejunostomy catheter is designed to be inserted following a laparotomy. The technique is a two part procedure with the catheter being first passed through the anterior abdominal wall, and then via the jejunal wall, into the lumen of the bowel.

DETAILED INSTRUCTIONS

Pass the distal end of the catheter through the conical attachment platform (fig 1). Select location for skin puncture, insert first introducer needle, withdraw needle leaving cannula in place. Pass catheter through the cannula into the abdominal cavity. Remove cannula from catheter by splitting the cannula along its length (fig 2). Make a loose purse string suture on the jejunal wall opposite the intra abdominal entry of the catheter. Do not remove the suture needle. Insert the second introducer needle through the centre of the suture. While stretching the jejunum below this point, advance the needle in the jejunal wall for approximately 10cm before entering the lumen of the bowel (fig 3). Verify the intra-jejunal position of the needle tip by injecting sterile water. Remove the introducer needle leaving the cannula in place. Pass the catheter through the cannula and position the catheter tip approximately 30cm into the jejunum (fig 4). Verify catheter position by injecting sterile water. Remove cannula from catheter by splitting the cannula along its length. Tighten the purse string suture around the catheter and in addition use 3 or 4 suture points to attach the jejunum to the intra abdominal wall at the catheter entry point (fig 5). Place the cutaneous fixing system in position and suture it to the skin. Tighten the locking screw and check that the catheter is firmly fixed (fig 6).

WARNING

BEFORE USE ENSURE THAT PLACEMENT IS STILL CORRECT BY X-RAY VERIFICATION USING CONTRAST MEDIUM VIA THE CATHETER. To prevent blockages flush with water after each use, and daily when not in use. Use liquid feeds (rather than powder formulations). Do not remove catheter before the 21st day after operation.

ADDITIONAL SKIN SUTURES ARE RECOMMENDED IN THE CASE OF RESTLESS PATIENTS.

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